# Uppackningsinstruktion

Gäller alla HD Balance 16 & 24

## Bilaga bruksanvisning

Sida 1 (1)

### VIKTIGT!

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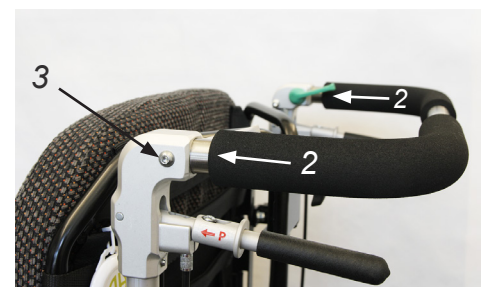
Page 1 (1)

### IMPORTANT!

The wheelchairs are going through a thorough final inspection before leaving the factory. Even though, a control of functions and a general review should be undertaken before the wheelchair is taken into use. This should be done to ensure that no damage has occurred during transportation.

- ▶ Always read the manual before the wheelchair is taken into use.
- ▶ Verify that all parts conform to the packing slip. Also note if the wheelchair is specially designed in any way (shown in the packing slip) and read carefully the documentation provided.
- ▶ The backrest of the wheelchair needs to be folded up and it is important that it is locked properly in an upright position. Please see the manual!
- ▶ If the push bar is removed at delivery, or if the wheelchair is equipped with separate operating handles, a mounting of these is required. Follow the instructions below.
- ▶ NOTE: Be careful not to damage the brake and control wires.

### Mounting of the push bar

- Loosen the screws (1) on the sides of the push bar.
- Insert the push bar into the holes on the brackets (2).
- Mount the screws on both sides (3).

Tools: Use a 4 mm hex key.

**NOTE:** Make sure that the screws are well tightened.

### Separate operating handles

The separate operating handles (4) can at delivery be folded against the backrest and the screws (5) are then packed in a plastic bag.

- Move the handles to the desired position. In each tube there is a threaded hole for locking the handles in the backrest frame.
- Tighten the screws with the washers to the backrest frame tube.
- There are three holes in the frame for different heights.
- Use the included tool that is placed under the wheelchair's seat on the right hand side.

**NOTE:** Make sure that the brake and control wires are running freely and not exposed to damage. The wires should run along the backrest frame.

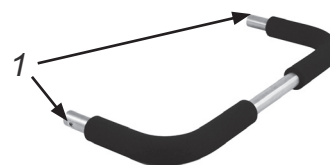
Diagram 1

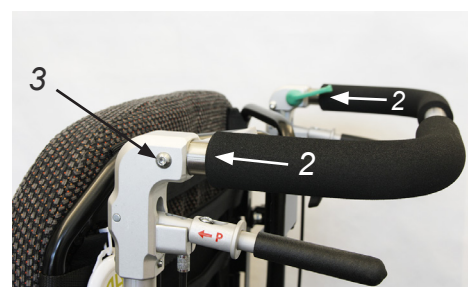
Diagram 2

Diagram 3