# Fotlåda till HD Balance

## Bruksanvisning / Monteringsanvisning

Sida 1 (1)

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1 x

### Montering på rullstol

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# Footbox for HD Balance

## User manual / Assembly instructions

Page 1 (1)

Parts included:

## Mounting on the wheelchair

To shift/mount the footbox on a HD Balance:

①

Image1. 4

Angle adjustable legrest

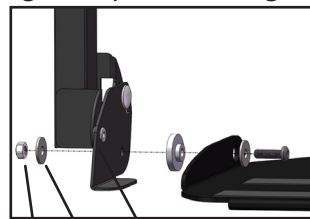
Image 2.

Fixed angle legrest

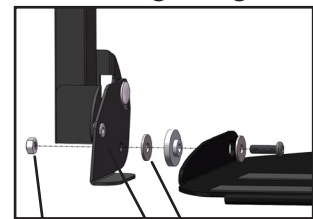
Image 3.

1. On delivery the footbox is prepared for an angle adjustable legrest. If it shall be mounted on a fixed angle legrest (90 or 105 degrees), a small adjustment is required:

- Release the nut (1) completely and dismount the washer (2) and the footplate holder (3) from the screw.
- Place the washer (2) between the footplate holder and the footplate instead and replace the nut again.

2. Release the screws for the calf support (4) and mount the footbox on the wheelchair as shown in image 4. Tighten the screws (4) when the footbox is in place.

3. Adjust the footbox to correct height.

4. Finished!

Image 4.

### If the footbox is hard to adjust (height)

- The plates that holds the calf support might be skew, this makes it hard to move the footbox.

To fix it: Remove the calf support (remove screws, 4) and bend the plates until they are in the same height and depth. It is recommended to do this while the footbox is mounted on the wheelchair!

## Use and Care instructions

The footbox can be cleaned by wiping with soap and water. It can also be disinfected with hand sanitizer. For further information, see the Maintenance Instructions, art. no. 95730-1.