

Kilformat sidostöd HD Balance


Art nr: 63440

Bruksanvisning / Monteringsanvisning


Sida 1 (1)

Ingående delar

Kilformat sidostöd bred

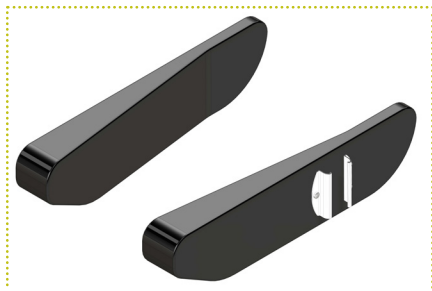


Kilformat sidostöd smal

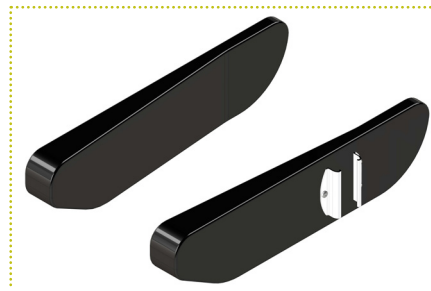

Montering

1

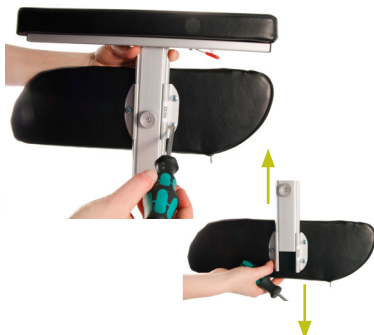

2



3



1. Demontera befintliga sidostöd genom att lossa lite på de tre skruvarna och dra isär armstödet och sidostödet.
2. Montera de kilformade sidostöden enligt bilden. Dra åt skruvarna.
3. Bilden visar de kilformade sidostöden monterade på stol.

Användning och skötselråd

De kilformade sidostöden kan rengöras genom avtorkning med tvål och vatten. För övrig information se Underhållsanvisning, art nr 95730.

Wedge side supports HD Balance


Art. no.: 63440

User manual / Assembly instructions

Page 1 (1)

Parts included

Wedge side supports Wide

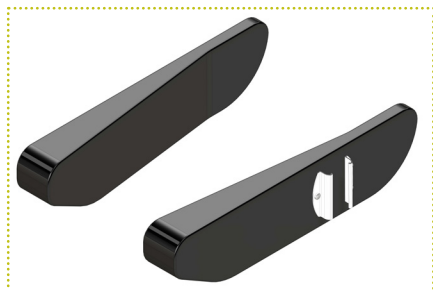

Wedge side supports Narrow



Assembly



1. Remove the current side supports by loosening the three screws and pulling the side support off the armrest.
2. Mount the wedge side supports as seen in the picture. Tighten the screws.
3. The picture shows the wedge side supports mounted on the chair.

Care instructions

The wedge side supports can be cleaned by wiping with soap and water. For further information see Maintenance Instructions, art. no. 95730-1.