

Armstödssarg HD Balance

Art nr: 62471 och 62472

Bruksanvisning / Monteringsanvisning

Sida 1 (1)

Ingående delar:

Sarg armstöd vänster, art nr: 62471




Sarg armstöd höger, art nr: 62472




Montering

①

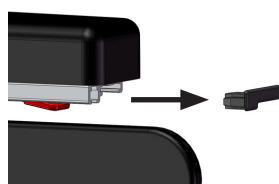

②

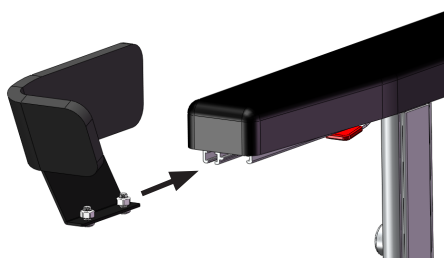

③



1. Dra ur den bakre plastpluggen ur armstödsskenan.
2. Montera skruvarna löst och trä i sargen i skenan från bakändan.
3. Sargen ska sitta så långt bak som möjligt. Dra åt skruvarna i önskad position.

Användning och skötselråd

Sargen kan rengöras genom avtorkning med tvål och vatten. För övrig information se Underhållsanvisning, art nr 95730.



- Om armarna av någon anledning vilar uppe på sargkanterna en längre tid kan skadligt tryck uppstå. Var uppmärksam på detta.

Elbowrest HD Balance

Art. no.: 62471 and 62472

User manual / Assembly instructions

Page 1 (1)

Parts included:

Elbowrest left, art.no: 62471




Elbowrest right, art.no: 62472




Assembling

①

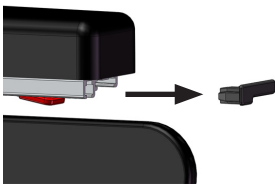

②

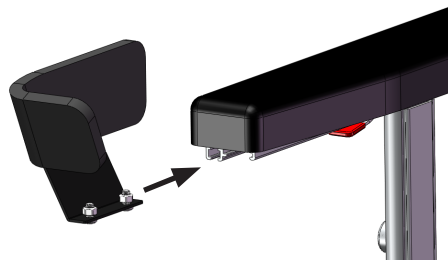

③



1. Pull out the rear plastic plug out of the armrest rail.
2. Mount screws loose and slide the elbowrest into the rail from the back.
3. The elbowrest should be placed as far back as possible. Tighten the screws in the desired position.

Use and care instructions

The elbowrest can be cleaned by wiping with soap and water. For further information see Maintenance Instructions, art. no. 95730-1.



- If the arms for some reason rest on top of the elbowrest edges for a longer period of time, harmful pressure can occur. Pay attention to this.