

Adapter - framflyttad sits HD Balance

Bruksanvisning / Monteringsanvisning


Sida 1 (1)

Ingående delar:



Montering

Instruktioner för att byta balanseringsplåtar finns i dokument 95720 - Teknisk information, avsnitt 2.4. Följ instruktionerna fram tills att timplåset är återmonterat. Efter detta ska adaptorn monteras enligt följande:

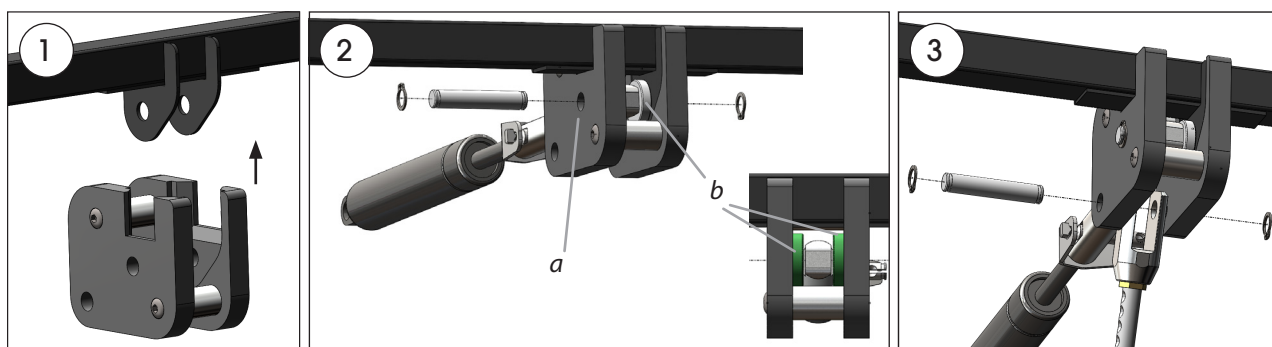

1. Ta först bort gasfjädern och tippstången från fästet i sitsramen och trä sedan på adaptorn. Skruvarna kan behöva lossas lite för att adaptorn ska gå att trä på.
2. Fäst in gasfjädern i det främre hålet (a). Använd en av tapparna, två spåringar och distansbrickorna (b) enligt bilden. Var alltid noga med att hantera spåringarna korrekt, defekta spåringar ska genast bytas ut. Skruva sedan åt skruvarna som håller ihop adaptorn.
3. Fäst tippstången i det bakre hålet med den andra tappan och spåringarna. Var alltid noga med att hantera spåringarna korrekt, defekta spåringar ska genast bytas ut.
4. Fortsätt sedan enligt instruktionerna i Teknisk information, avsnitt 2.4.

Adapter, moved forward seat

User manual / Assembly instructions


Page 1 (1)

Parts included:



Assembling

Instructions on how to change the tilt plates is found in document 95720-1, Technical Information, section 2.4. Follow the instructions until the tilt lock is remounted. Thereafter mount the adapter according to the following:

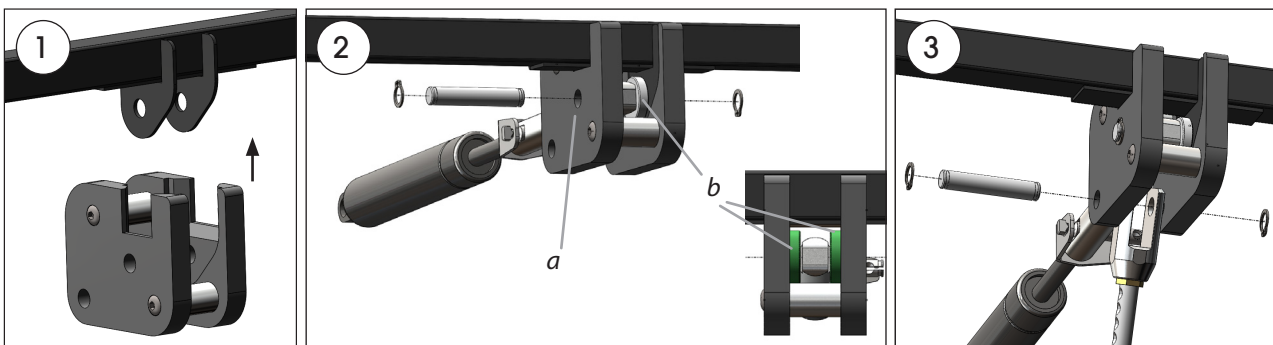

1. First remove the gas spring and the tilt bar from the attachment point at the seat frame. Then attach the adapter according to the picture. The screws may need to be loosened a little if the adapter is too tight to get onto the frame.
2. Fix the gas spring in the front hole (a). Use one of the pins, two circlips and the spacers (b), according to the picture. Then tighten the screws of the adapter. Always handle circlips with care and replace damaged ones immediately.
3. Attach the tilt bar to the rear hole with the remaining pin and circlips. Always handle circlips with care and replace damaged ones immediately.
4. Continue according to the instructions in document 95720-1, section 2.4.