Ryggdistanser, HD Balance

Bruksanvisning / Monteringsanvisning

Sida 1 (1)

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Back spacers, HD Balance

User manual / Assembly instructions

Page 1 (1)

Intended use


Back spacers are used to achieve a shorter seat depth than otherwise possible. When using HD Balance back spacers the seat depth is decreased with 2,5 cm. The use implies risks, see below, and the back spacers are marked with warning labels:


- The use of back spacers impact wheelchair balancing and should only be used in exceptional cases. The center of gravity moves and makes it more prone to tipping forward.
- Back Spacers should be used only for patients who really need the short seat depth.
- Patients with a behavior that might tip the wheelchair forward can combine the back spacers with a backward positioned seat unit. See info sheet 95942-1 for more info.

Parts included:

4 x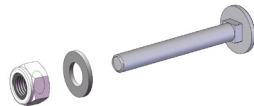

Assembling

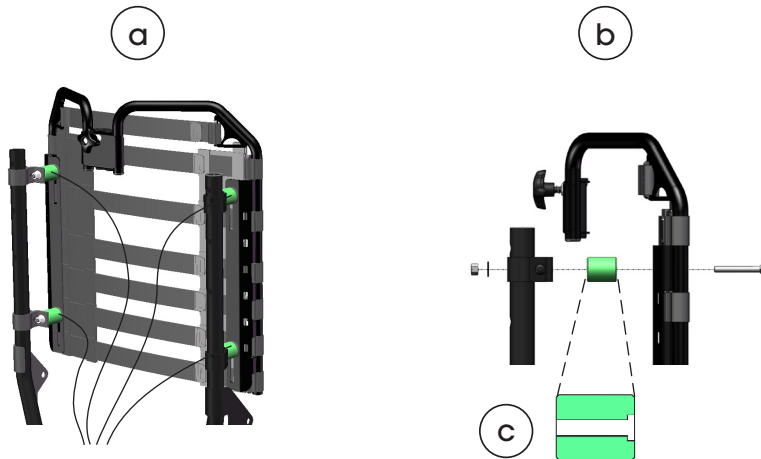1. Remove the back system from the brackets.
2. Mount the 4 spacers between the brackets and the back system with the supplied bolts, washers and nuts according to (a) and (b). It is **important** that the countersink points forward (c).
3. Finished!

Use and care instructions

Can be cleaned by wiping with soap and water. For further information, see the Maintenance Instructions, art. no. 95730-1.