


# Adapter bålstödsfäste

## - För HD Balance med smalt ryggsystem


### Bruksanvisning / Monteringsanvisning

Sida 1 (1)

#### Ingående delar:

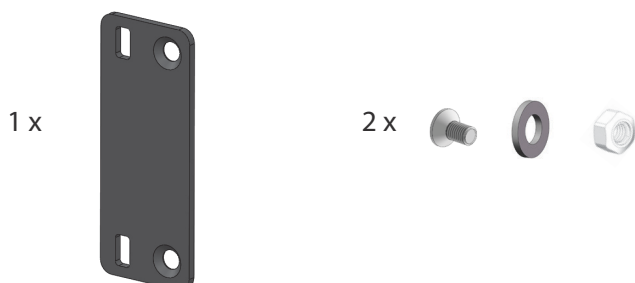

#### Montering

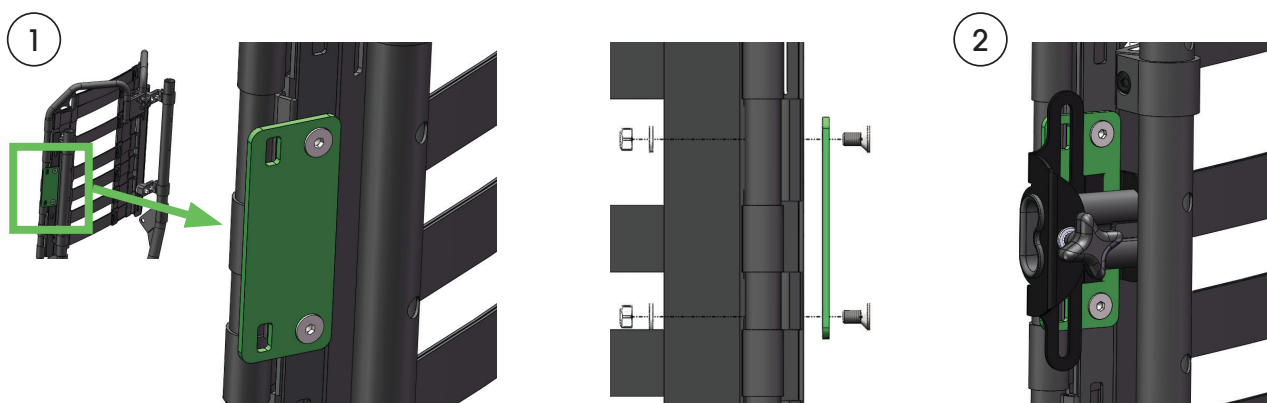

1. Placera adapterplåten på önskad höjd längs ryggens sida och passa in mot lämpliga hål. Fäst med de medföljande skruvarna enligt bilderna ovan.

2. Montera därefter bålstödsfästet i de två spåren på adapterplåten. Skruvar till detta medföljer bålstödsfästet.

#### Användning och skötselråd

Används för att möjliggöra montering av bålstöd på HD Balance utrustad med smalt ryggsystem (-4cm). Bålstöden hamnar ca 20mm längre ut åt sidan jämfört med ett ryggsystem i standardbredd.


Torkas av med tvål och vatten vid behov.

# Adapter thorasic support bracket - HD Balance with narrow back system


## User manual / Assembly instructions

Page 1 (1)

### Parts included:

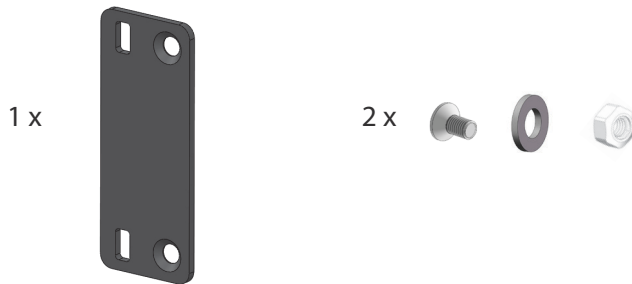

### Assembling

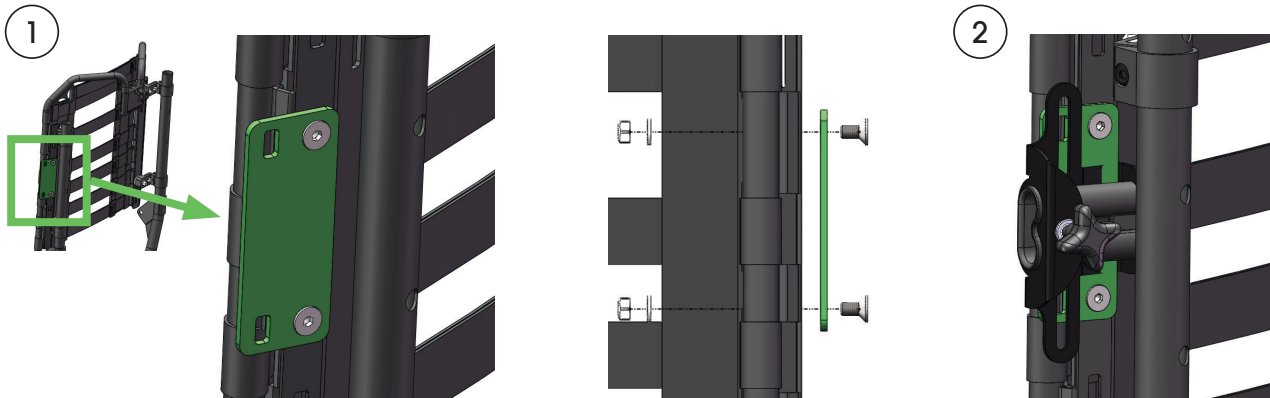

1. Position the adapter plate in the desired height along the side of the back frame. Choose suitable holes and mount with the supplied screws.
2. Mount the thorasic support bracket in the two slots on the adapter plate. Screws are supplied together with the thorasic support.

### Use and Care instructions

The adapter makes it possible to use thorasic supports on HD Balance wheelchairs that have a narrow back system (-4cm). The thorasic supports will be positioned about 20mm wider compared to a back system with standard width.

Can be cleaned by wiping with soap and water. It can also be disinfected with hand sanitizer. For further information, see the Maintenance Instructions, art. no. 95730-1.