


Avsedd användning


Sidostöd kil används som alternativ till standard sidostöd för HD Balance rullstolar.

Ingående delar

- 1 x sidostöd kil
(7° vänster modell visas)

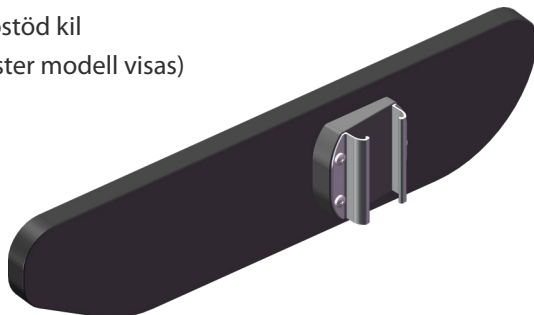

Montering

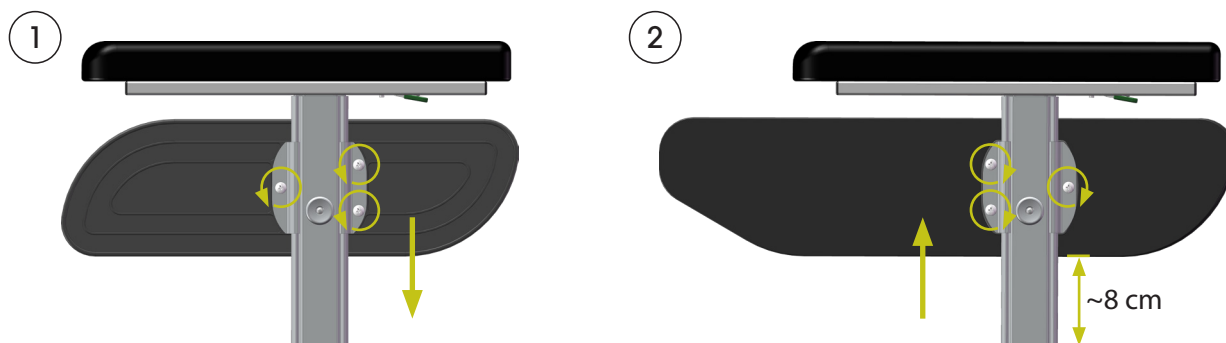

- ① Demontera befintliga sidostöd genom att lossa lite på de tre skruvarna och dra isär armstödet och sidostödet.
- ② Skjut sidostöd kil på plats på armstödssolpen. Stolpens längd under sidostödet ska vara ungefär 8 cm. Dra åt skruvarna.

Användning och skötselråd


Sidostöd kil kan rengöras genom avtorkning med tvål och vatten. Byt alltid ut skadade delar omedelbart. För övrig information se *Underhållsanvisning*, dokument nummer 95730.

Intended use

Wedge side support is used as an alternative to the standard side support for HD Balance wheel chairs.



Parts included

1 x wedge side support
(7° left hand model shown)

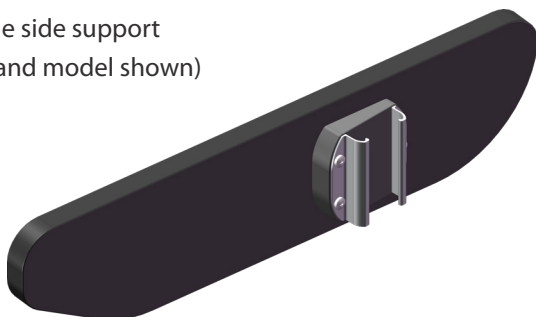

Assembly

①

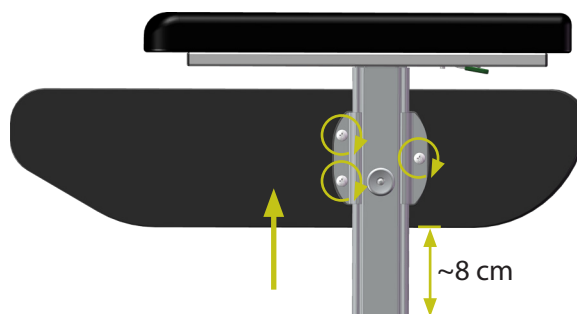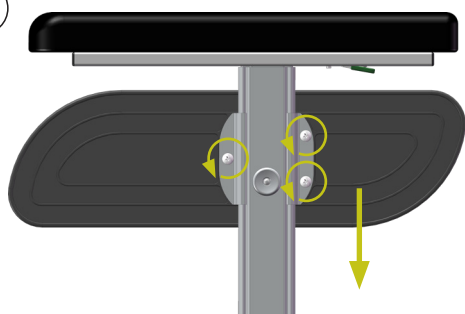

- ① Remove the current side support by loosening the three screws and pulling it down off the armrest post.
- ② Push the wedge side support into place on the armrest post. The length of the post under the support should be approximately 8 cm. Tighten the three screws..

Use and Care Instructions

Wedge side supports can be cleaned by wiping off with soap and water. Always replace damaged parts immediately. For more information see *Maintenance Instructions*, document number 96730-1.