

Knäled - frikopplingsbar

HD Balance

Art nr: 62733-5 och 62734-5

Bruksanvisning / Monteringsanvisning

Sida 1 (1)

Ingående delar:

62733-5 Knäled frikopplingsbar vänster



62734-5 Knäled frikopplingsbar höger




Montering

Denna knäled monteras på samma sätt som den vinkelställbara knäleden utan frikopplingsreglage.

Användning

Följ nedanstående instruktioner vid användning av frikopplingsbar knäled.

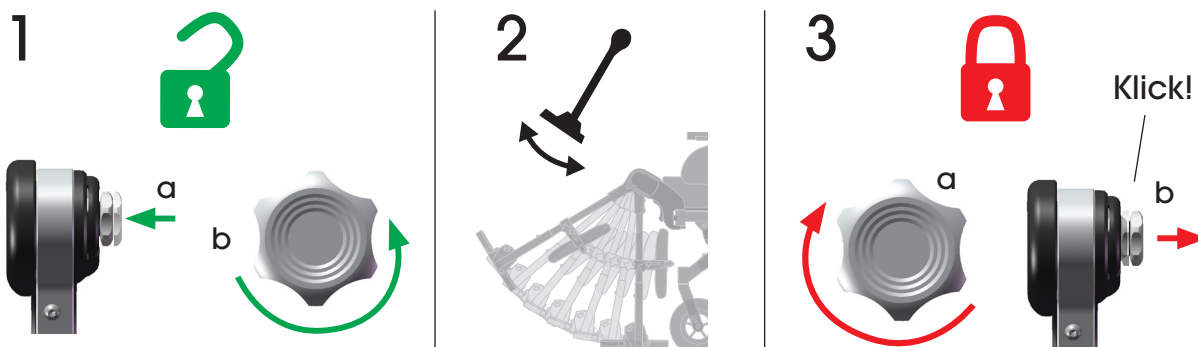

1 (a, b) Tryck på vredet och vrid samtidigt moturs. Släpp sedan vredet. Nu är denna knäled upplåst.

2. Reglera ben- och fotstöd genom att på vanligt sätt låsa upp knäleden på andra sidan.

⚠ Håll alltid emot ordentligt i benstöden vid denna reglering. När vredet är upplåst faller ben- och fotstöd ner direkt när knappen på den vanliga knäleden trycks in.

3 (a, b). Gå tillbaka till den frikopplingsbara knäleden. Se till så att båda knälederna är inställda på samma vinkel. Vrid vredet medurs. Dra lite i benstödet så att vredet klickar ut till sitt yttre läge.

Skötselråd

Torkas av med tvål och vatten vid behov. För ytterligare underhållsinformation, se Underhållsanvisning 95730.



- Knäleden ska alltid vara låst förutom vid justering av benstödsvinkeln. Om den lämnas upplåst kan konstruktionen skadas.

Knee joint - releasable

HD Balance

Art. no.: 62733-5 and 62734-5

User manual / Assembly instructions

Page 1 (1)

Parts included:

62733-5 Knee joint releasable left



62734-5 Knee joint releasable right




Mounting

This knee joint is mounted in the same manner as the standard knee joint.

Use

Follow the instructions below when using the releasable knee joint.

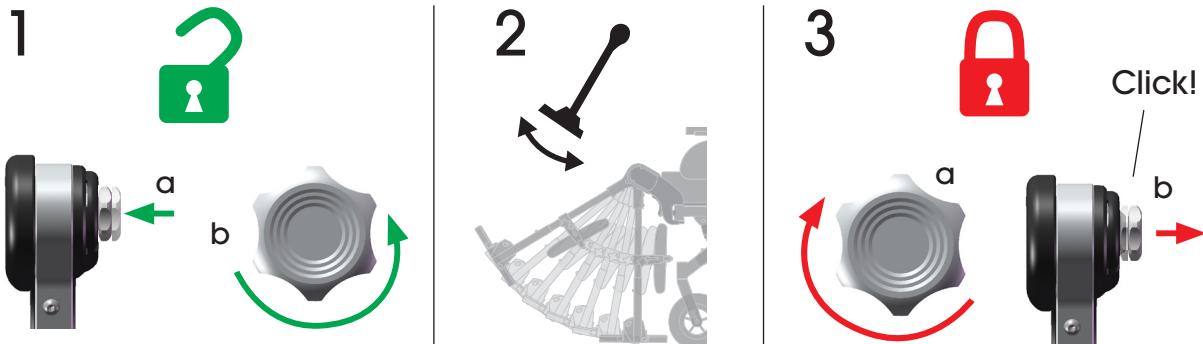

1 (a, b) Push and turn the knob counterclockwise. Now the knee joint is unlocked.

2. Adjust the leg- and footrest by unlocking the opposite knee joint in the normal way.

 Always hold the legrest securely when doing this adjustment. When the knob is unlocked the leg- and footrest fall down immediately when the button on the standard knee joint is pushed.

3 (a, b). Return to the releasable knee joint. Make sure that both knee joints are set at the same angle. Turn the knob clockwise. Pull the legrest a bit so that the knob clicks out to its outer position.

Care instructions

Wipe with soap and water when needed. For further information, see the Maintenance Instructions 95730-1.



- The knee joint shall always be locked except when adjusting the angle of the legrests. If it's left unlocked, the structure can be damaged.