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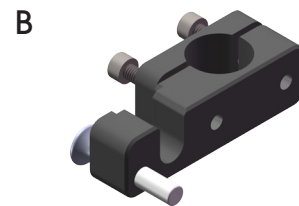
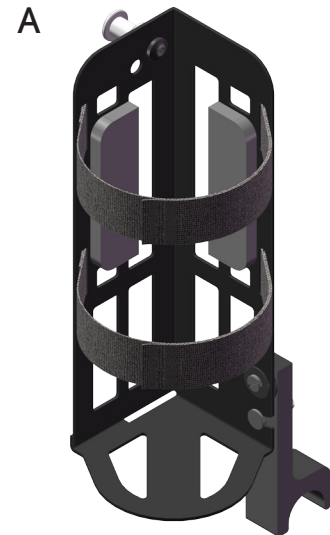
When supplemental oxygen is needed HD Balance can be equipped with an oxygen tank holder.

Oxygen tank holders are available in two models. The smaller model (A) is suited for 1 L oxygen tanks and the larger model (B) is suited to oxygen tanks of 2 to 2,5 L. The oxygen tanks used should be of the type LIV from AGA AB, or equivalent.

Mount for oxygen tank holder

The oxygen tank holder is held in place via a mount (C) which is attached to the back frame of the wheelchair. The oxygen tank is then safely and easily secured in the holder with two velcro straps.

In use

The oxygen tank holder with tank can easily be removed from the wheelchair to remain with the user when transferring out. The foot of the holder rotates up, so that the holder can stand on a flat surface.

Measurements

The height of the oxygen tank holder is 26 cm.

The maximum diameter for the oxygen tank is 12 cm.

Keep in mind

- Follow instructions from the manufacturer of the oxygen tank regarding proper handling thereof.
- If the oxygen tank holder and tank are ever placed on the floor, be aware of the risk for tipping.
- Using an oxygen tank holder with tank increases the risk of tipping for the wheelchair, particularly when the seat is tilted and the back reclined.
- Use anti-tip devices.

Article numbers

Article numbers for the oxygen tank holders and mount are found in the Prescription form, document number 95750-1.