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It is possible to equip HD Balance with an IV pole. This can make it easier for the user to be more mobile for certain types of treatments or examinations.


Construction is stable and consists of a strong, light-weight aluminium pole (A), and a mount (B) which is attached to the back frame of the wheelchair on the left or right side. The length of the IV pole is easily adjusted with a knob. The angle is adjusted with a knob so that the IV pole can always be held upright regardless if the seat or back are tilted. The pole has two hooks so that it can carry two drip bags.

B

The IV pole mount is labelled with a warning symbol on the outside to highlight that special information exists. See document 95707-1.

A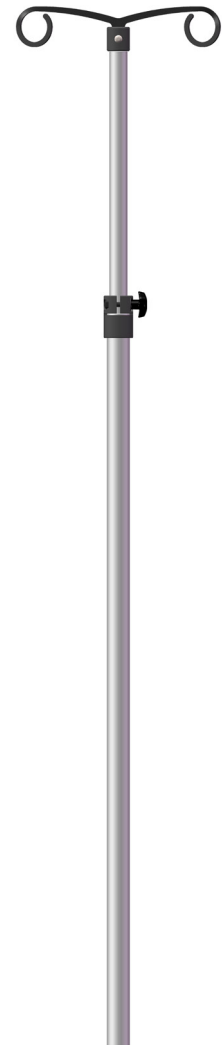

Keep in mind

- Always keep the IV pole as vertical as possible.
- Always check that the knob is tightened before use, otherwise the IV pole can fall forwards or backwards.
- Using an IV pole increases tipping risk, especially with heavy drip bags and with the seat or back tilted.
- Be aware of doorway heights when an IV pole is used.
- Always use anti-tip devices.

Article numbers

All article numbers are found in the Prescription Form, art.nr. 95750-1.