

Avsedd användning

Knä/Bendelarfäste används för att montera knädelare alternativ bendelare på HD Motion.

Ingående delar

1 x fäste



4 x skruv


MFX M5 x 10



Montering

Ta loss sittdynan. Notera sittbrickans position på skalan som finns på höger sidan, flytta sedan sittbrickan till sin yttersta position.

Skruva fast fästet med medföljande skruvar enligt bilden. Återställ sedan sittbrickan till sin ursprungliga position och sätt tillbaka sittdynan.

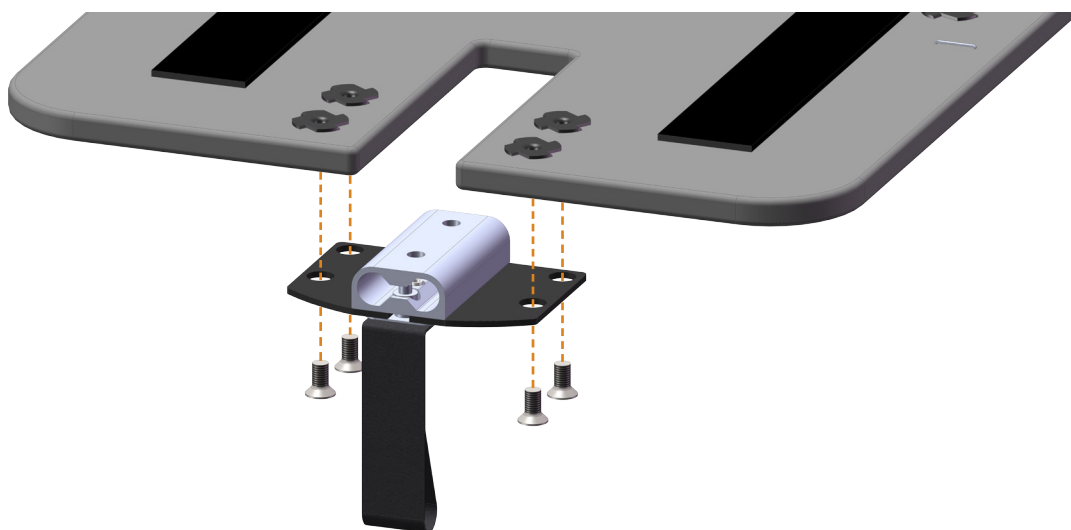

Användning och skötselråd

Byt alltid ut skadade delar omedelbart. För övrig information se Underhållsanvisning, dok. nr. 96730.

Intended use

Spacer mount is used to mount a Knee Spacer or Leg Spacer on HD Motion.

Parts included

1 x mount



4 x screw


MFX M5 x 10



Assembly

Remove the seat cushion. Note the position of the seat plate per the scale on the right side, then move the seat plate to the outermost position.

Screw the spacer mount in place with the included screws as shown. Return the seat plate and the seat cushion to their original positions.

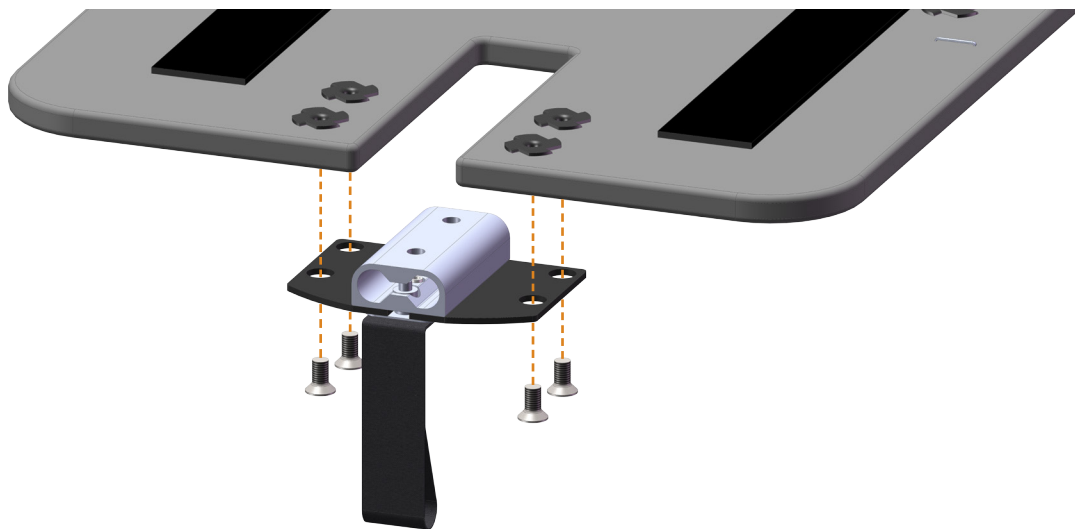

Use and Care instructions

Always replace damaged parts immediately. For more information see *Maintenance Instructions*, document number 96730-1.