


Avsedd användning


Ryggdistanser används för att åstadkomma ett kortare sittdjup än vad som annars är möjligt.

Ingående delar

4 x distanser

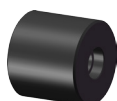

4 x bult MVBF M6

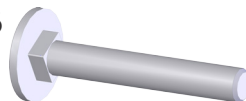


4 x fläsmutter M6




Montering

Ta loss ryggdynan, sittdynan och armstöd.

- 1 Demontera ryggsystemet från ryggklamrarna.

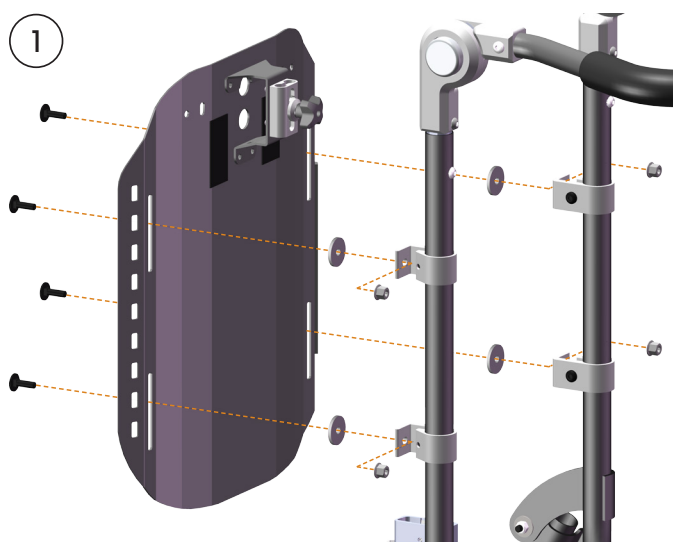

- 2 Montera de 4 ryggdistanserna mellan ryggklammer och ryggsystem med de medföljande bultarna och muttrarna.

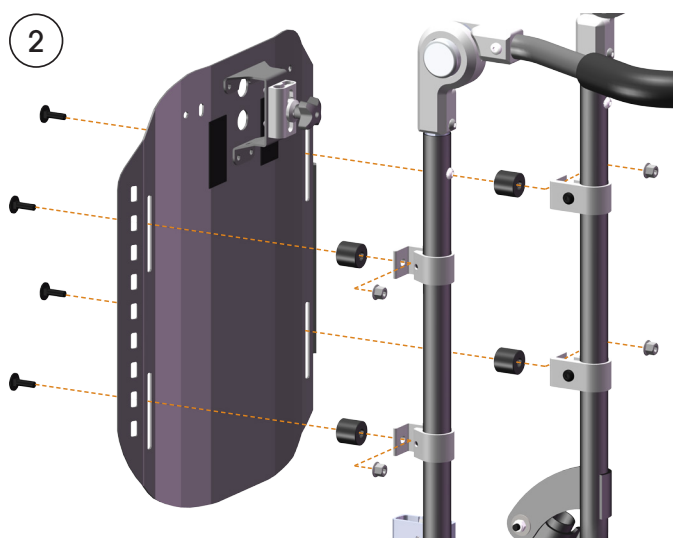

Användning och skötselråd


Byt alltid ut skadade delar omedelbart. För övrig information se Underhållsanvisning, dok nr 96730.

Intended use

Back spacers are used to create a shorter seat depth than is otherwise possible.

Parts included

4 x spacer

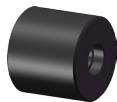

4 x bolt MVBF M6




4 x flange nut M6




Assembly

Remove back cushion, seat cushion, and armrests.

- 1 Remove the backrest system from the backrest brackets.

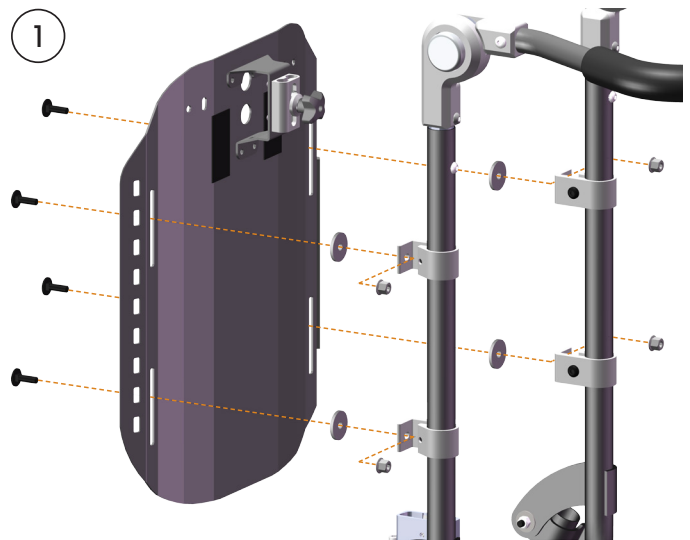

- 2 Assemble the 4 back spacers between the backrest clamps and the backrest with the included bolts and nuts.

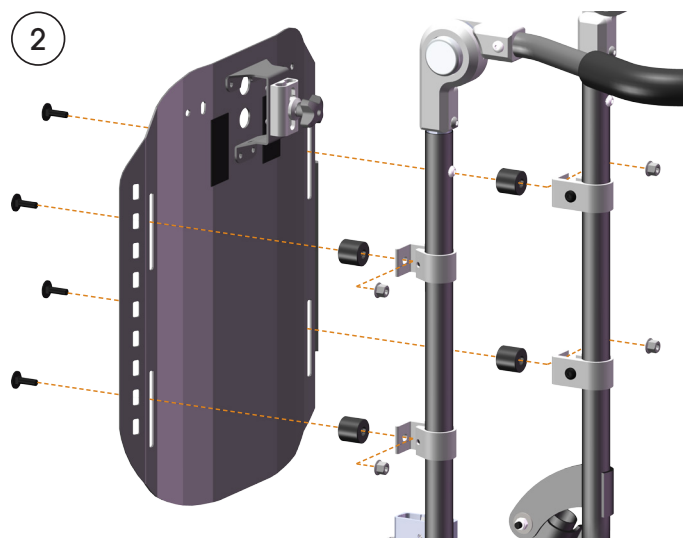

Use and Care instructions

Always replace damaged parts immediately. For more information see *Maintenance Instructions*, document number 96730-1.